

Collection Procedure for Confirmations to the Lab

1. After obtaining a positive result on the Instant Drug Test, send the specimen into the laboratory for confirmation.
2. Locate the long security seal that came with the chain of custody. Have the donor initial the security seal. Collector fills out the donor ID# on the seal (the donor ID# must match the ID# on the chain of custody). Collector signs and dates.
3. Place the security seal over the specimen.
4. Open the lab bag.
5. Place the specimen and the lab's copy of the completed chain of custody into the lab bag provided. Make sure the bag is sealed.
6. Put the bag into the provided lab pak with the pre-paid shipping label on the outside of the pak.
7. Call for pick up.